

Installation Procedure of the Matrox Drivers for the Vascular Imager

Medical Imaging Applications LLC
832 Forest Hill Drive
Coralville, Iowa 52241, USA
Web: <http://www.mia-llc.com>
Email: support@mia-llc.com
Phone: 319-358-1529
Fax: 319-688-5296

Prerequisite:

1. The Matrox PCI frame grabber has to be installed on the computer, before installing the Matrox drivers.
2. If any previous version of Matrox drivers (earlier than version 7.0) is installed, this Matrox driver has to be un-installed first. If you have any Brachial Imager or Vascular Imager version earlier than 4.0 installed and functional, then you already have Matrox drivers installed and you must uninstall them).

To uninstall previous Matrox driver:

- a) Go to the **Control Panel**.
- b) Launch “**Add or Remove Programs**”.
- c) Select “**Matrox Imaging Products**” and remove it. If you are prompted to re-start your computer, please do so or the un-installation may fail.

To install new Matrox drivers:

Insert the Vascular Tools Installation CD, provided by MIA, in the CD-ROM drive of the computer with Matrox Meteor-II PCI frame grabber board installed.


The new Matrox drivers are included on the Vascular Tools Installation CD. Please, follow these steps:

- 1) Please insert the Vascular Tools Installation CD.
- 2) This will start the installation process of Vascular Tools. Assuming that you have already installed the Vascular Tools software, please "cancel" the installation.
- 3) Go to "My Computer" and "Open" the CD with Vascular Tools installed. This may start the installation process again - if it does, please cancel it again.
- 4) Navigate to the **Utilities** folder on the CD, find the **MeteorII-7.5** folder and open it.
- 5) You shall see a **setup.exe** or **setup** file - double click on that.
- 6) Follow the instructions detailed on the next pages.


Medical Imaging Applications LLC
Web: <http://www.mia-llc.com>
Email: support@mia-llc.com
Page 1 of 5

Steps:


1. Select **Redistribution - ActiveMIL** from the installation window, click “**Continue**”

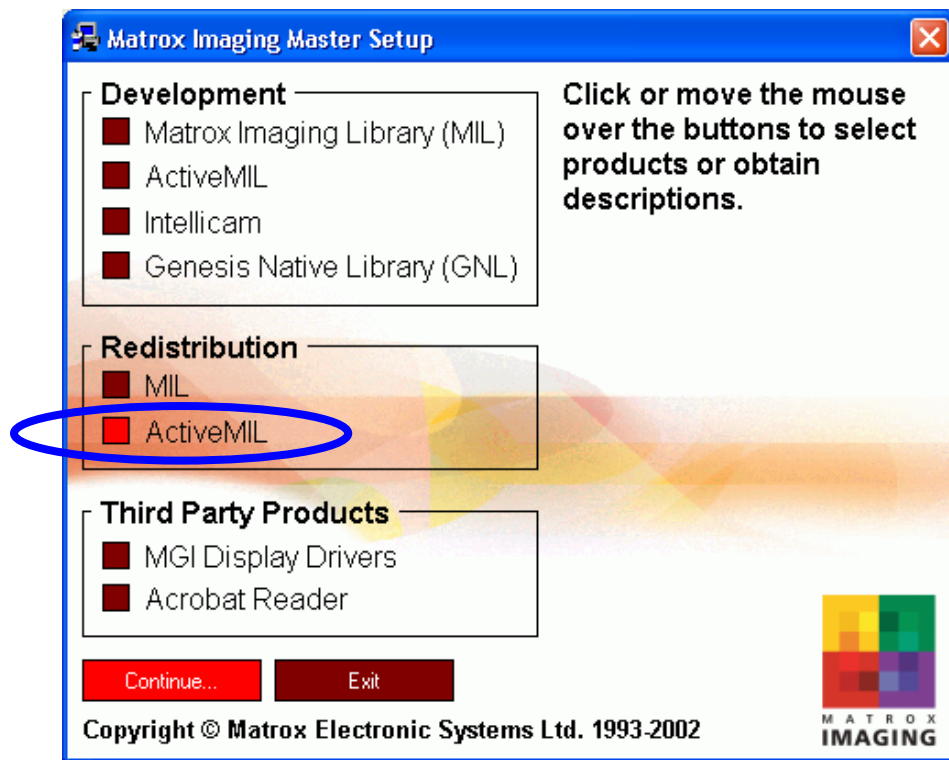


2. Accept all default options in subsequent windows, until you are prompted to select the board drivers to install. Check **MeteorII/Standard** and **VGA** from the list. And click on the **Next** button.

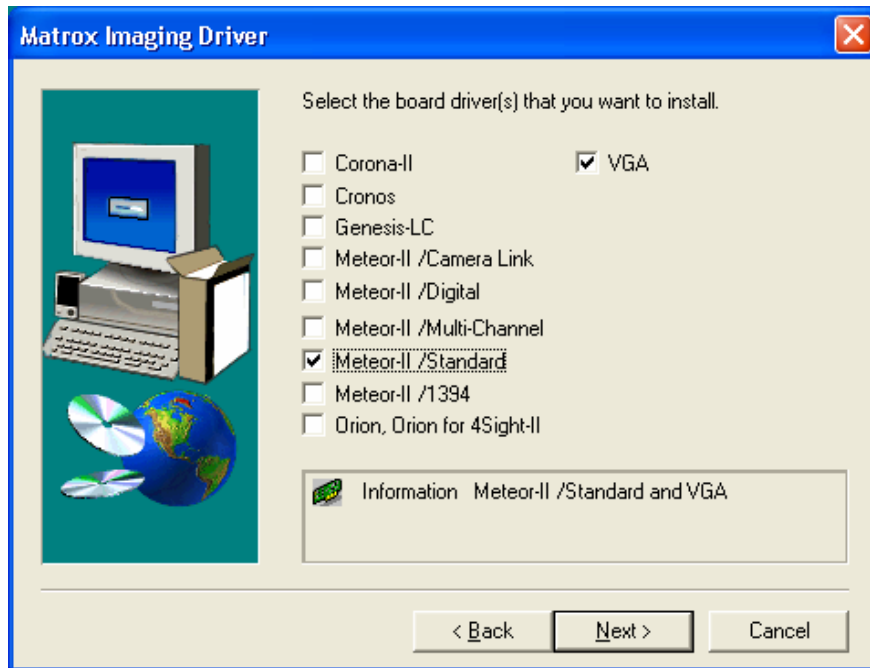


3. In the next screen you are asked whether you have a floppy disk labeled “MIL Driver ...”, check **No** and click on the **Next** button.

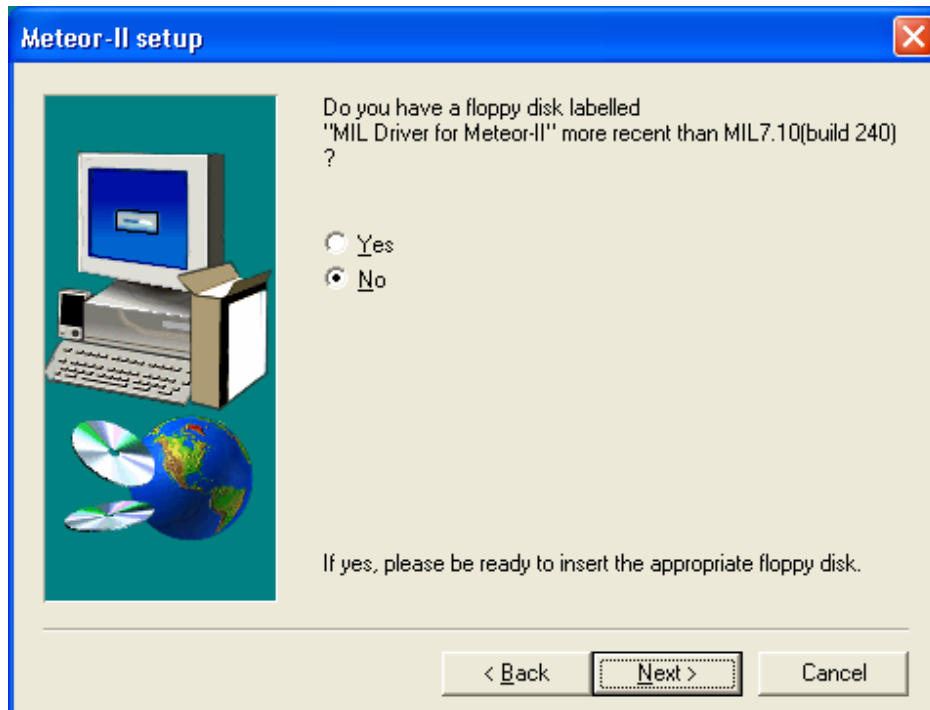

4. In the next window you are asked whether you have a floppy disk labeled “MIL Driver for VGA”, check the **No** and click on the **Next** button.

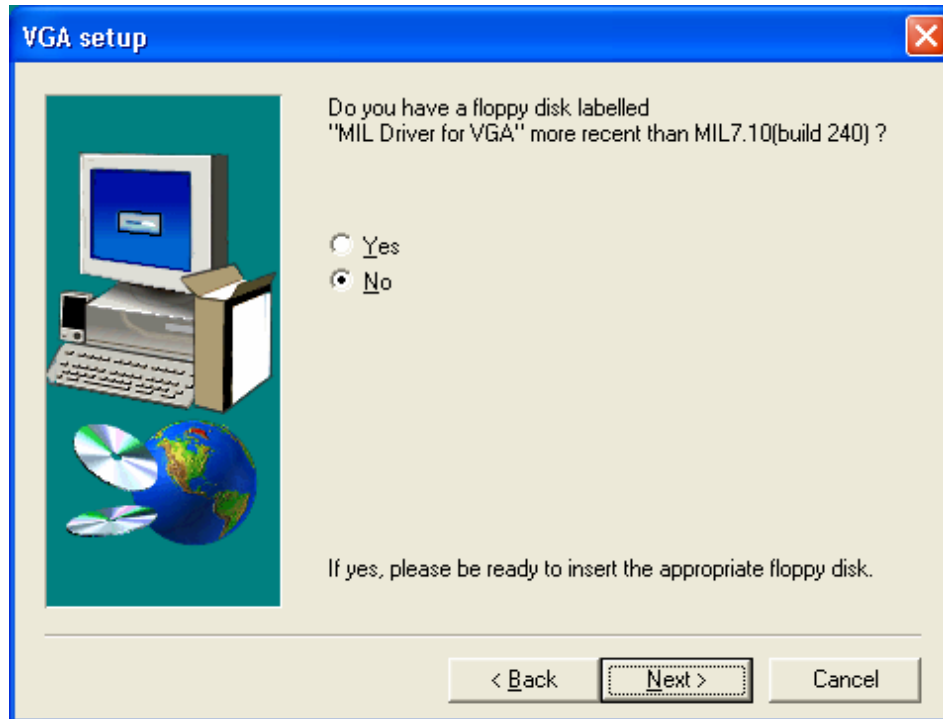

5. In the next window about “MIL DMA Manager”, take the default **4MB** box checked. Click on the **Next** button.

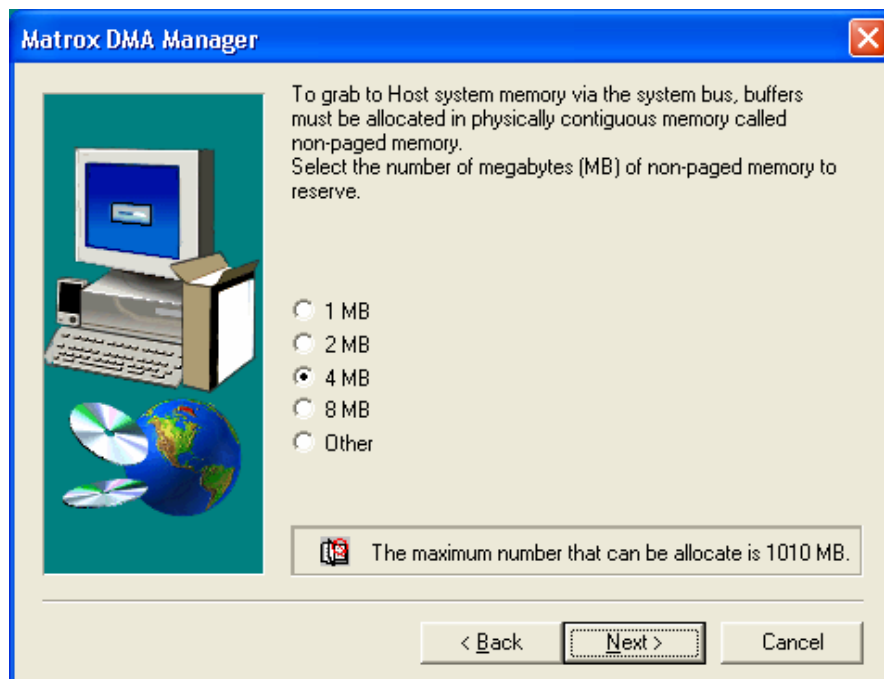

6. Take defaults and click on Next in the next couple windows. The last window may ask you to re-start the computer. Please make sure the “**Yes, I want to restart my computer now**” is checked. Click on Finish button to re-start. **You**

must follow this very important step to re-start your computer or the installation may fail and the Vascular Imager may not function.

